Description:
ESPUMA® With Exulock technology Family (Adhesive and Non-Adhesive) is absorbent polyurethane foam dressing with uniformly distributed highly absorbing powder.
The outermost layer is breathable, bacteria-proof, virus-proof and waterproof polyurethane film that protects the wound from external contamination.
Upon absorption of exudates, the foam matrix forms a soft gel that allows easy removal of the dressing.

Intended Purpose:
ESPUMA® With Exulock technology Foam dressings are indicated for chronic or acute wounds with moderate to heavy exudates such as:

- Pressure ulcers, venous leg ulcers and diabetic foot ulcers.
- Surgical wounds, donor sites, traumatic wounds, malignant wounds, graft sites.
- First and second degree burns.
- Suitable for use under compression bandaging.

- The dressing should not be used on dry wounds, third degree burns, dry eschar, sinus tracts and wounds with excessive bleeding.
- Sterility of the dressing is guaranteed unless the pouch is damaged or opened prior to use.
- Do not cover the dressing with occlusive films or tapes, as they may reduce moisture vapor loss, which could impair the effectiveness of the dressing.
- The dressing should not be used in combination with oxidizing agents such as hydrogen peroxide or hypochlorite solution as these solutions may affect the structure and performance of the polyurethane foam.
- At the time of dressing change, if the dressing is adhered to the wound surface, saturate with normal saline.
- Consult a healthcare professional if there are signs of infection or sensitivity (allergic reaction).
- Reuse of the dressing may lead to increased risk of infection or cross contamination.

1. Cleanse the wound in accordance with local clinical protocols.
2. Select an appropriate dressing size that overlaps the wound margin by at least two centimeters.
3. Remove the sterile dressing from the package using aseptic technique.

1. Non-Adhesive variants can be cut to fit awkward areas. When used under compression therapy, the dressing must be cut to the size of the wound.

3. Secondary film dressing or conforming bandage should be used to secure the dressing in place.

1. Remove the central paper window (if present).
2. Remove the large flap of the release liner to expose the adhesive portion of the dressing.
3. Position into place while removing the smaller flap of the release liner of the dressing.
4. Slowly remove the paper frame while smoothing down around the dressing to ensure good adhesion.
5. To remove the dressing, loosen the border before lifting the dressing away from the wound.

EC REP
E C Rep Ltd,
5 Fitzwilliam Square East,
Dublin 2, D02 R744, Ireland
Tel: +353 1 2 544 944
Email: info@ecrep.ie

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Visit our website

 Amria 25612, Alexandria - Egypt www.pharmaplast-online.com			Leaflet artwork template no: I-TM /AR/001-5 (Version 5)				
Customer : Pharmaplast			Product name : Espuma Evulcol family				
Issue date : 16-11-2016			Prepared by : Marina Medhat				
Colors : 			Dimension : 49x15.5cm				
Issue Number	Revision Date	Description of Change	Rationale For Change	Amended by	Effective date	Approved by	
						Artwork manager	RA Department
1	N/A	New Issue	N/A				
2	12/09/2018	1-Adding test prepared 2- adding EC Rep symbol 3-Adding manufacturer symbol	Essential requirement				
3	17/10/2018	new temperature symbol	Essential requirement				
4	07/04/2019	- Removing store in cool dry place and add new temperature symbol -New recycling symbol -Open book symbol	To be comply with ISO 15223-1. Due to the brevit issues.				
5	08/05/2019	amend text by Jostina	To be aligned with CER				
6	24/06/2019	Adding multi languages	Customers needs				
7	06-07-2019	Text amendments	Product name translated				
8	19-08-2019	amendment in family name	to be the same as the Technical file				
9	2/01/2020	amending typos	leaflet maintenance				
10	01/06/2020	- Adding languages - Adding symbols translation - Modifying the design to adapt the addition process.	- Marketing point of view				
11	06-04-2022	- Adding new recycle symbol - Adding new caution symbol - Adding new store in dry and cool place symbol - Adding new keep away from direct sunlight symbol - Removing laptop symbol from address	file maintenance				